

Work 'n' Play Nested Tables

INSTRUCTIONS FOR USE

Codes 2445 & 2446




CONTENTS	PAGE
1.0 INTRODUCTION	1
2.0 ILLUSTRATION OF YOUR WORK 'N' PLAY NESTED TABLES	2
3.0 FOR YOUR SAFETY	3
4.0 UNPACKING AND ASSEMBLING YOUR PRODUCT	3
5.0 HOW TO USE YOUR PRODUCT	4
6.0 CARE & MAINTENANCE	5
7.0 GUARANTEE & SERVICE	5
8.0 CONTINUOUS IMPROVEMENT	6
9.0 WARRANTY & AFTERSALE	6

1.0 INTRODUCTION

Thank you for choosing the Smirthwaite Work 'n' Play Nested Table set.


This product has been designed to provide a versatile and durable table set which can be neatly stored on its own mobile base when not in use.

The tables can be used individually or in combination – the smallest table can be used as a footrest (or an aid to help transition from crawling to standing) and the remaining sizes can be used either as bench style seats or standalone tables to provide a support surface for activities.

	<p>IMPORTANT!</p> <p>These instructions should be read by all therapists and carers using the equipment and should be retained for future reference. The product should always be used under adult supervision.</p> <p>Any incorrect use of the product and failure to follow the instructions may put the user at risk or impede the function. If you have any queries using this product or wish for further copies, please do not hesitate to contact Customer Service department on T: +44 (0)1626 835552</p>
---	--

2.0 ILLUSTRATION OF YOUR WORK 'N' PLAY NESTED TABLES


- A. Table top
- B. Mobile base
- C. Size identifier

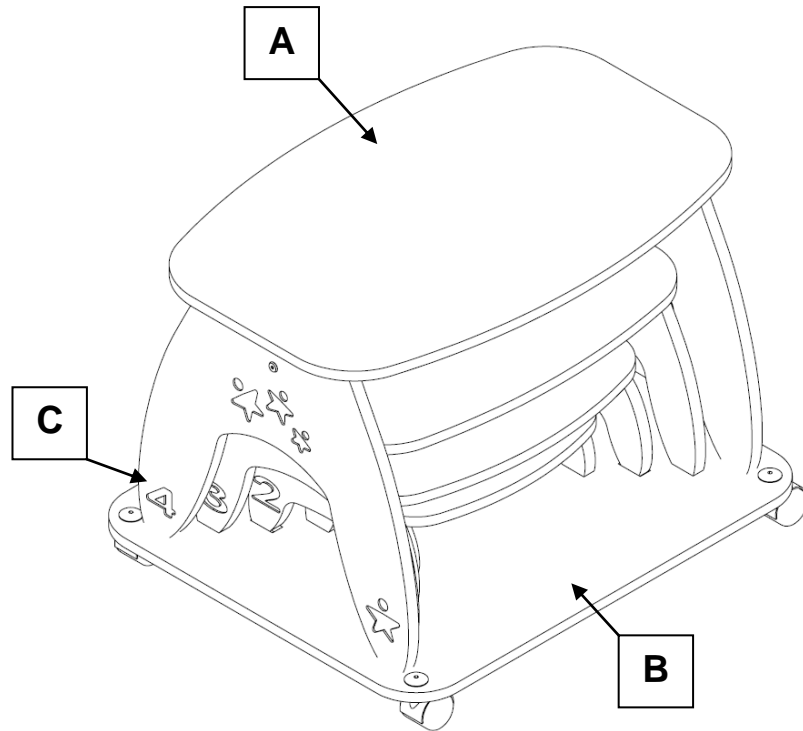

2.1 TECHNICAL DATA

Size	Height (mm)	Length (mm)	Depth (mm)	Max User Weight (kg)
1	110	355	266	80
2	170	460	320	80
3	290	565	375	80
4	450	720	460	80

3.0 FOR YOUR SAFETY

	STOP!
	Please read these instructions CAREFULLY and THOROUGHLY


- The user should NOT be left unattended whilst using the product. Always ensure a responsible therapist or carer is in attendance.
- If you believe the product to be faulty at any time, **DO NOT USE** – contact Smirthwaite by telephone on +44 (0)1626 835552.
- The product is **ONLY** to be used indoors on a flat level surface.
- **DO NOT USE any of the tables or the mobile base as an adult step or transit device.**
- Regular maintenance checks and cleaning are essential for the safe use of this equipment (see Section 6.0 Care and Maintenance).
- Always keep this product away from naked flames, cigarettes and sources of heat including open fireplaces, radiators, heaters.
- **DO NOT** fit parts or accessories of other manufacturers to this product unless authorised to do so in writing by G&S Smirthwaite Ltd. Failure to follow these instructions will not only invalidate the guarantee but could make the product dangerous to use. G&S Smirthwaite Ltd will not accept liability for any injury or damage incurred through such malpractices. Any repairs required must be carried out by G&S Smirthwaite Ltd authorised personnel.

4.0 UNPACKING AND ASSEMBLING YOUR PRODUCT

- Your product will be delivered either flat-packed for self-assembly (Code 2446) or fully assembled (Code 2445)
- Please refer to the assembly guide for details of how to assemble the flat-packed variant.

If you believe this product to be faulty **-DO NOT USE-**Contact G&S Smirthwaite Ltd on T: +44 (0)1626 835552

5.0 HOW TO USE YOUR PRODUCT


	STOP!
	The user should NOT be left unattended whilst using the product. Always ensure a responsible therapist or carer is in attendance.

- The Work 'n' Play set can be used in a variety of combinations to assist in therapy sessions/activities.
- For example, when using Size 4 table as a support surface it can be combined with Size 3 table as a seating bench and Size 1 table as a foot rest (shown below).



- Alternatively, each table can be used independently as either a support surface or bench, with a maximum weight loading of 80kg per table.
- The tables are also ideal for use as steps to practice ascending and/or descending.
- When the tables are not in use the mobile base provides a convenient storage solution, minimizing storage space required.


6.0 CARE AND MAINTENANCE

	IMPORTANT!
	Cleaning is recommended on a regular basis

- Clean wooden parts with a damp cloth and mild detergent
- Stubborn marks on wood work should be cleaned by using a soft brush.
- Do not soak or immerse the product in water.
- Store the product in a cool dry place out of direct sunlight.
- DO NOT use bleach, solvents, abrasives, synthetic detergents, wax polishes, antibacterial sprays or wipes.
- For further information please refer to MHRA or your NHS cleaning guidelines.

6.1 DAILY CHECKS

- Check all parts for signs of wear and tear or damage.
- Ensure all screws are present and tightened

	STOP!
	The product should not be in use while the checks are carried out.

6.2 SERVICE INTERVAL

Work 'n' Play tables should be serviced every 3 years. Servicing must only be undertaken by a Smirthwaite service engineer, or by a Smirthwaite trained representative.

7.0 GUARANTEE & SERVICE

The product is issued with a full parts and labour guarantee for 2 years from the date of delivery. This guarantee does not apply to accidental damage caused through inappropriate use of the product. This guarantee is issued at the discretion of G & S Smirthwaite Ltd on RTB (Return to Base) basis. If you have any questions or doubts relating to the safety or use of the product please contact us for advice or assistance on:

T: +44 (0)1626 835552

E: enquiries@smirthwaite.co.uk

8.0 CONTINUOUS IMPROVEMENT

G & S Smirthwaite Ltd are committed to continuous improvement to their product range. Should you have any suggestions or comments please send them to our product design department at: enquiries@smirthwaite.co.uk

G&S Smirthwaite Ltd reserve the right to change the specification or material without prior notice.

For catalogues, help and further information on our products please contact us at:

G&S Smirthwaite Ltd 17 Wentworth Road Heathfield, Devon.TQ12 6TL

T: +44 (0)1626 835552

F: +44 (0)1626 835428

E: enquiries@smirthwaite.co.uk

W: www.smirthwaite.co.uk

9.0 WARRANTY & AFTERSALE

9.1 Product Information

Model:	
Size:	
Date of Manufacture:	
Serial Number:	
Final Inspection:	

9.2 Service & inspection record form:

Date	Procedure	Service Personnel

CE



G & S Smirthwaite, 17 Wentworth Road, Heathfield, Newton Abbot, Devon.TQ12 6TL
T: +44 (0) 1626 835552 F: +44 (0) 1626 835428 E: enquiries@smirthwaite.co.uk
www.smirthwaite.co.uk